


Straumann iEXCEL™ – BLC/BLX Healing Abutment selection

QUICK GUIDE

The total height of a Healing Abutment for BLC/BLX consists of two parts:

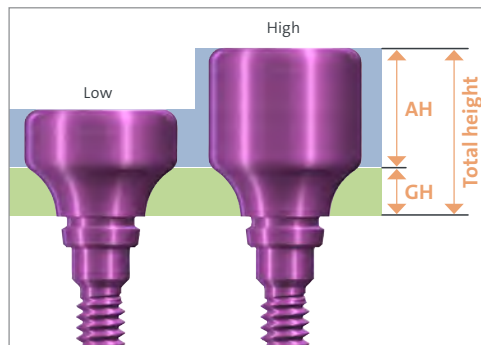

Gingiva Height (GH):

Emerging part of the healing abutment, available in heights of 0.75 mm to 3.5 mm. Uniform profile throughout the prosthetic portfolio.

Abutment Height (AH):


Cylindrical part of the healing abutment, available in three heights - 2 mm (low), 4 mm (medium) or 6 mm (high).

Always communicate the **implant diameter** and the selected **healing abutment** to the dental lab!

Please consider, that the selected Healing Abutment determines the available options for the final restoration.

Slim profile single crown

RB/WB Healing Abutment

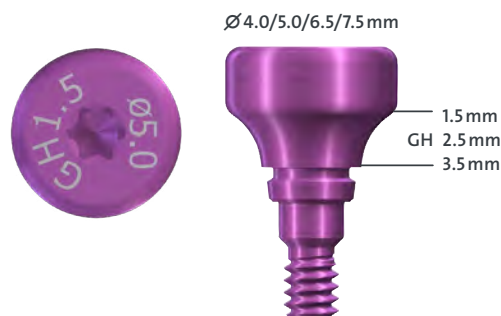


After healing, no wide profile crown (WB) nor bridge restoration possible, see on the back page.

Note: Use Healing Abutment Ø 5.0, GH 1.5 mm for Variobase® AS and RB/WB CARES® Abutments.

Wide profile single crown, molar


WB Healing Abutment

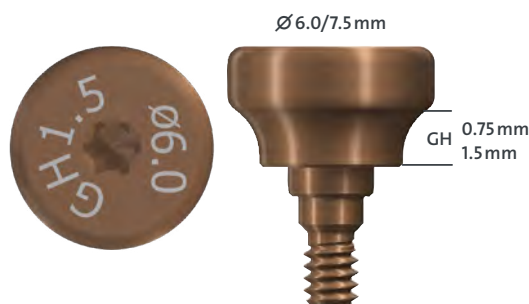

Note: For WB BLC/BLX implants.

Bridge


RB/WB Healing Abutment for Bridge/Bar

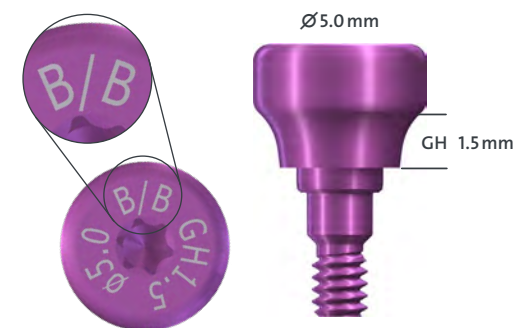


Note: Use this healing abutment if it has not yet been determined whether the restoration will consist of single crowns or a bridge.

For more information refer to the Straumann BLX™ Implant System Consistent Emergence Profiles™ and Healing Abutment selection (704659/en).

Single crown: WB Variobase® cannot be seated!

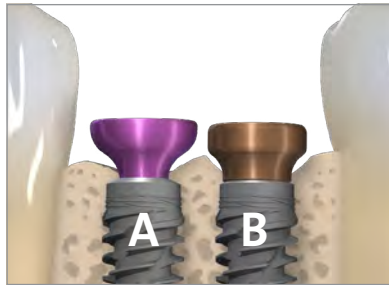

A Implant with a RB/WB Healing Abutment (magenta) has been used

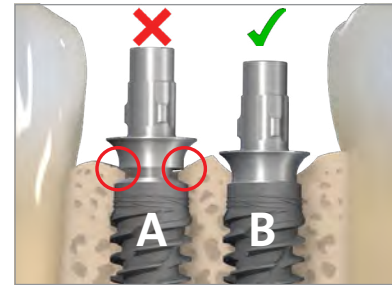


B Implant with a WB Healing Abutment (brown) has been used



A WB Variobase® cannot be seated properly due to bone covering the implant shoulder

A WB Variobase® can be seated since the implant shoulder was properly protected by the appropriate WB Healing Abutment during the healing phase


RB/WB Healing Abutment for Single Crown

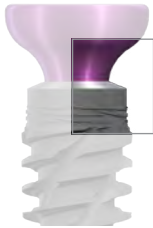

- Emerges from Ø 2.9 mm
- No protection of outer shoulder



WB Healing Abutment for Bridge/Bar

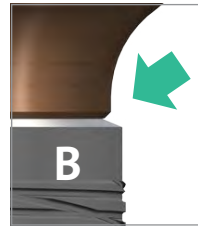

- Emerges from Ø 4.0 mm
- Protection of outer shoulder of WB implants



International Headquarters

Institut Straumann AG
 Peter Merian-Weg 12
 CH-4002 Basel, Switzerland
 Phone +41 (0)61 965 11 11
 Fax +41 (0)61 965 11 01
www.straumann.com

© Institut Straumann AG, 2025. All rights reserved.

Straumann® and/or other trademarks and logos from Straumann® mentioned herein are the trademarks or registered trademarks of Straumann Holding AG and/or its affiliates.